



## **SYLOPLASTER LIFTOPLASTER**

## **SYLOPLASTER**

Dispenser containing up to 30 kg of plaster. The inside rubber container is sealed against humidity to avoid the spoiling of plaster or investment. When the pouring diaphragm is opened, a vibrating device starts to operate thus ensuring a uniform flowing of the powder without stones avoiding the plaster clogging





## **LIFTOPLASTER**

Lifter to be fixed on the wall to easily move the SYLOPLASTER up and down, to reach respectively the filling or the working position.





remove the top



remove the basket



open the carter



Lift down the container



remove the cover



pour the plaster



remove the old bag



open a new bag



fix the new bag



fix the cover

To keep the plaster or the investment dry, it is possible to fix inside the container a product absorbing the humidity.

When the product is transformed from powder into a gel, it is the right time to replace it to save plaster, investment and money.



lift up the container